





## Tightening Torque Values

| Abutment                             | Abutment Screw            | Compatible Reflect Implants                              | Compatible OEM Implants   | Tightening Torque, Ncm |
|--------------------------------------|---------------------------|--|---|------------------------|
| <b>Reflect Aspire</b>                |                           |  |   |                        |
| Aspire Cover Screw Ø 3.5 mm          | NA                        | Reflect Aspire<br>Body Ø 3.5 mm<br>Platform Ø 3.5 mm     | Astra Tech OsseoSpeed™<br>Body Ø 3.5 mm<br>Platform Ø 3.5/4.0 mm      | 5-10                   |
| Aspire Ø 3.5/4.0 mm Healing Abutment | NA                        |  |   | 5-10                   |
| Aspire Ø 3.5 mm 30° Abutment         | Aspire Screw 3.5/4.0      |  |   | 35                     |
| Aspire Cover Screw Ø 4.0 mm          | NA                        | Reflect Aspire<br>Body Ø 4.0 mm<br>Platform Ø 4.0 mm     | Astra Tech OsseoSpeed™<br>Body Ø 4.0 mm<br>Platform Ø 3.5/4.0 mm      | 5-10                   |
| Aspire Ø 3.5/4.0 mm Healing Abutment | NA                        |  |   | 5-10                   |
| Aspire Ø 4.0 mm 30° Abutment         | Aspire Screw 3.5/4.0      |  |   | 35                     |
| Aspire Cover Screw Ø 5.0 mm          | NA                        | Reflect Aspire<br>Body Ø 5.0 mm<br>Platform Ø 5.0 mm     | Astra Tech OsseoSpeed™<br>Body Ø 5.0 mm<br>Platform Ø 4.5/5.0 mm      | 5-10                   |
| Aspire Ø 4.5/5.0 mm Healing Abutment | NA                        |  |   | 5-10                   |
| Aspire Ø 5.0 mm 30° Abutment         | Aspire Screw 4.5/5.0      |  |   | 35                     |
| <b>Reflect □</b>                     |                           |  |   |                        |
| Precise Cover Screw Ø 3.4 mm         | NA                        | Reflect Precise<br>Body Ø 3.4 mm<br>Platform Ø 3.4 mm    | Biomet 3i Certain®<br>Body Ø 3.25 mm<br>Platform Ø 3.4 mm             | 10                     |
| Precise Ø 3.4 mm Healing Abutment    | NA                        |  |   | 20                     |
| Precise Ø 3.4 mm 30° Abutment        | Precise Screw 3.4/4.1/5.0 |  |   | 20                     |
| Precise Cover Screw Ø 4.1 mm         | NA                        | Reflect Precise<br>Body Ø 4.1 mm<br>Platform Ø 4.1 mm    | Biomet 3i Certain®<br>Body Ø 4.0 mm<br>Platform Ø 4.1 mm              | 10                     |
| Precise Ø 4.1 mm Healing Abutment    | NA                        |  |   | 20                     |
| Precise Ø 4.1 mm 30° Abutment        | Precise Screw 3.4/4.1/5.0 |  |   | 20                     |
| Precise Cover Screw Ø 5.0 mm         | NA                        | Reflect Precise<br>Body Ø 5.0 mm<br>Platform Ø 5.0 mm    | Biomet 3i Certain®<br>Body Ø 5.0 mm<br>Platform Ø 5.0 mm              | 10                     |
| Precise Ø 5.0 mm Healing Abutment    | NA                        |  |   | 20                     |
| Precise Ø 5.0 mm 30° Abutment        | Precise Screw 3.4/4.1/5.0 |  |   | 20                     |
| <b>Reflect Rapid</b>                 |                           |  |   |                        |
| Rapid Cover Screw NP                 | NA                        | Reflect Rapid<br>Body Ø 3.5 mm<br>Platform NP (Ø 3.5 mm) | NobelActive® and NobelReplace Conical<br>Body Ø 3.5 mm<br>Platform NP | 5-10                   |
| Recover/Rapid NP Healing Abutment    | NA                        |  |   | 10-20                  |
| Rapid NP 30° Abutment                | Recover/Rapid Screw NP    |  |   | 35                     |
| Rapid Cover Screw RP                 | NA                        | Reflect Rapid<br>Body Ø 4.3 mm<br>Platform RP (Ø 3.9 mm) | NobelActive® and NobelReplace Conical<br>Body Ø 4.3 mm<br>Platform RP | 5-10                   |
| Recover/Rapid RP Healing Abutment    | NA                        |  |   | 10-20                  |
| Rapid RP 30° Abutment                | Recover/Rapid Screw RP    |  |   | 35                     |
| Rapid Cover Screw RP                 | NA                        | Reflect Rapid<br>Body Ø 4.3 mm<br>Platform RP (Ø 3.9 mm) | NobelActive® and NobelReplace Conical<br>Body Ø 5.0 mm<br>Platform RP | 5-10                   |
| Recover/Rapid RP Healing Abutment    | NA                        |  |   | 10-20                  |
| Rapid RP 30° Abutment                | Recover/Rapid Screw RP    |  |   | 35                     |

| Abutment                          | Abutment Screw         | Compatible Reflect Implants                                | Compatible OEM Implants   | Tightening Torque, Ncm |
|-----------------------------------|------------------------|--|---|------------------------|
| <b>Reflect Recover</b>            |                        |  |   |                        |
| Recover Cover Screw NP            | NA                     | Reflect Recover<br>Body Ø 3.5 mm<br>Platform NP (Ø 3.5 mm) | NobelActive® and<br>NobelReplace Conical<br>Body Ø 3.5 mm<br>Platform NP                            | 5-10                   |
| Recover/Rapid NP Healing Abutment | NA                     |  |   | 10-20                  |
| Recover NP 30° Abutment           | Recover/Rapid Screw NP |  |   | 35                     |
| Recover Cover Screw RP            | NA                     | Reflect Recover<br>Body Ø 4.3 mm<br>Platform RP (Ø 3.9 mm) | NobelActive® and<br>NobelReplace Conical<br>Body Ø 4.3 mm<br>Platform RP                            | 5-10                   |
| Recover/Rapid RP Healing Abutment | NA                     |  |   | 10-20                  |
| Recover RP 30° Abutment           | Recover/Rapid Screw RP |  |   | 35                     |
| Recover Cover Screw RP            | NA                     | Reflect Recover<br>Body Ø 5.0 mm<br>Platform RP (Ø 3.9 mm) | NobelActive® and<br>NobelReplace Conical<br>Body Ø 5.0 mm<br>Platform RP                            | 5-10                   |
| Recover/Rapid RP Healing Abutment | NA                     |  |   | 10-20                  |
| Recover RP 30° Abutment           | Recover/Rapid Screw RP |  |   | 35                     |
| <b>Reflect Tapered</b>            |                        |  |   |                        |
| Tapered Cover Screw Ø 3.5 mm      | NA                     | Reflect Tapered<br>Body Ø 3.7 mm<br>Platform Ø 3.5 mm      | Tapered Screw-Vent®<br>Body Ø 3.7 mm / Platform<br>Ø 3.5 mm<br>Body Ø 4.1 mm / Platform<br>Ø 3.5 mm | 5-10                   |
| Tapered Ø 3.5 mm Healing Abutment | NA                     |  |   | 10-20                  |
| Tapered Ø 3.5 mm 30° Abutment     | Tapered Screw 3.5/4.5  |  |   | 35                     |
| Tapered Cover Screw Ø 3.5 mm      | NA                     | Reflect Tapered<br>Body Ø 4.1 mm<br>Platform Ø 3.5 mm      | Tapered Screw-Vent®<br>Body Ø 4.7 mm<br>Platform Ø 4.5 mm   | 5-10                   |
| Tapered Ø 3.5 mm Healing Abutment | NA                     |  |   | 10-20                  |
| Tapered Ø 3.5 mm 30° Abutment     | Tapered Screw 3.5/4.5  |  |   | 35                     |
| Tapered Cover Screw Ø 4.5 mm      | NA                     | Reflect Tapered<br>Body Ø 4.7 mm<br>Platform Ø 4.5 mm      | Tapered Screw-Vent®<br>Body Ø 6.0 mm<br>Platform Ø 5.7 mm   | 5-10                   |
| Tapered Ø 4.5 mm Healing Abutment | NA                     |  |   | 10-20                  |
| Tapered Ø 4.5 mm 30° Abutment     | Tapered Screw 3.5/4.5  |  |   | 35                     |



**Manufactured for ids**  
 145 Cedar Lane  
 Englewood NJ, 07631

Tel. 866-277-5662 / 201-676-2456  
 For details, visit [www.idsimplants.com](http://www.idsimplants.com)

Symbols Glossary

ANSI/AAMI/ ISO 15223-1:2012 Medical devices – Symbols to be used with medical device labels, labeling and information to be supplied – Part 1: General requirements.

| Symbol         | Title of Symbol<br>(Reference Number)  |
|----------------|--|
| <b>i</b>       | Consult Instructions for Use<br>(5.4.3)  |
|                | Caution: Consult accompanying documents<br>(5.4.4)   |
|                | Manufacturer<br>(5.1.1)  |
|                | Date of manufacture<br>(5.1.3)   |
|                | Use-by date<br>(5.1.4)   |
|                | Batch code<br>(5.1.5)  |
|                | Catalogue number<br>(5.1.6)  |
|                | Serial number<br>(5.1.7)   |
|                | Sterilized using irradiation<br>(5.2.4)  |
|                | Do not re-use<br>(5.4.2)   |
|                | Do not re-sterilize<br>(5.2.6)   |
|                | Do not use if package is damaged<br>(5.2.8)  |
|                | Temperature limitation<br>(5.3.7)  |
| <b>Rx Only</b> | For USA Only (Federal law(USA) restricts this device to sale by or on the order of a licensed dentist) |
|                | CE conformity marking  |
|                | Authorized representative in the European community<br>(5.1.2)   |